REF FS 692

INSTRUKCJA OBSŁUGI
UŽIVATELSKÝ MANUÁL
BRUGSANVISNING
BENUTZERHANDBUCH
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
USER MANUAL
KASUTUSJUHEND
KORSNIČKE UPUTE
FELHASZNÁLÓI ÚTMUTATÓ
MANUALE D'USO
NAUDOJIMO INSTRUKCIJA
LIETOTĀJA INSTRUKCIJA
MANUAL DO UTILIZADOR
MANUAL DE INSTRUȚIUNI
NÁVOD NA POUŽITIE
NAVODILA ZA UPORABO
BRUKSANVISNING

PL INSTRUKCJA OBSŁUGI	4
CS UŽIVATELSKÝ MANUÁL	9
DA BRUGSANVISNING	13
DE BENUTZERHANDBUCH	17
EL ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	22
EN USER MANUAL	27
ET KASUTUSJUHEND	32
HR KORSNIČKE UPUTE	36
HU FELHASZNÁLÓI ÚTMUTATÓ	40
IT MANUALE D'USO	44
LT NAUDOJIMO INSTRUKCIJA	49
LV LIETOTĀJA INSTRUKCIJA	53
PT MANUAL DO UTILIZADOR	57
RO MANUAL DE INSTRUCTIUNI	62
SK NÁVOD NA POUŽITIE	67
SL NAVODILA ZA UPORABO	71
SV BRUKSANVISNING	75

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Kod produktu: FS 692

1. Wykonanie

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Product code: FS 692

1. Characteristics

The transport commode chair is made of a powder-coated steel frame. Flip-up armrests help with transfers of the patients with significantly reduced mobility. The chair features a soft seat upholstered with waterproof, easy-to-clean material, four swivel castors with brakes, swing-away footrests and a toilet bucket. Thanks to its special design, the chair can be positioned over a toilet for convenient use.

The set includes: frame, seat, backrest, footrests, protective strap, toilet bucket with lid, assembly key for footrest.

Elements of the commode chair:

- A. Locking clips of the frame
- B. Seat
- C. Armrest
- D. Locking pin
- E. Footrest
- F. Assembly pin of the footrest
- G. Castor lock
- H. Toilet bucket

2. Application

Transport commode chair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. The chair is suitable for indoor use only. The chair can be handled only with the help of an accompanying person. The chair functions as a transport-toilet chair thanks to the toilet bucket located under the seat and four swivel castors. It facilitates the user everyday hygiene. The chair is especially recommended to people who have problems with mobility which are caused by:

- paralysis,

- overweight or abnormal underweight,
- degenerations or deformity of limbs,
- degenerations or joints injuries,
- limbs injuries or amputations,
- heart and blood vessels disease, circulatory disorders.

3. Assembly

Step 1. Remove locking clips located on the back frame.

Step 2. Slide the backrest tubes into the openings of the main frame and secure it with locking clips afterwards.

Step 3. Assemble the armrests in the front part of the frame till you hear click sound.

Step 4. Assemble the right and the left footrest in the front of the frame by placing them onto two pins and turning them inwards until they are locked.

Step 5. Assemble and adjust the protective strap on the footrests.

Step 6. Slide from the back the toilet bucket (without the lid) into the rails located under the seat.

Step 7. Check if commode chair is correctly assembled and is not missing any parts.

The wheelchair is ready to use.

WHEELCHAIR HANDLING

- Check the condition of the castor brakes, frame and wheels. Also check all assembled components and their adjustments.
- Before use, ensure that the chair's width (45 cm) is suitable for the user, as the chair is not width-adjustable.
- The chair is intended for short-distance use indoors only. It is not equipped with shock absorption, so prolonged use is not recommended.

Note! Do not use the chair if any defects or irregularities are detected.

FLIP-UP FUNCTION OF ARMRESTS

- Press the locking pin and move armrest upwards.
- To close the armrest, move it downwards until you hear “click” sound of the locking pin.
- **Do not** use the armrest as a support while standing up. If required, ask the third person for help.

BRAKES

- To lock the castors, press the lever of the right and left lock with foot. To release the castor, pull the lever upwards using foot.

FOOTRESTS

- The footplates should be in vertical position when getting in or out the wheelchair.
- **Do not use the footplates as a support while getting in or out the wheelchair due to the risk of wheelchair tipping over and falling.**

GETTING IN THE WHEELCHAIR

- While getting in or out the wheelchair, changing the users position or stopping, the castors must be locked.
- The wheelchair should be located as close as possible to a place of changing a seat. The wheelchair should touch the bed or other seat. If the user cannot change their seat independently, caregiver or third person has to transfer of the user to the wheelchair.
- The user should sit as deep as possible in the middle of the seat in order to avoid chair instability.

4. Technical data

Overall width	55 cm
Inner width	45 cm
Overall depth	87 cm
Seat height from the ground	53 cm
Overall height	95 cm
Seat dimensions	44 x 44 cm
Weight	10,60 kg
Maximum capacity	100 kg
Folded width	55 cm
Frame color	white

5. Notes

Timago International Group does not bear responsibility of improper moving in the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.
- Empty and disinfect the toilet bucket and toilet seat after each use. Use cleaning agents according to producer's recommendations.

7. Storage and transport conditions

The product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction

The manufacturer checked for compliance with essential medical devices requirements.

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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